OPERATION MANUAL

Rx Only

MADE IN JAPAN

OM-DC0927EN 001

Contra Angle Handpiece Titanium Body

Ti-Max X

X55L / X55

CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the handpiece always consider the safety of the patient.
- The handpiece is designed only for dental clinical use by licensed person.
- Do not attempt to disassemble the handpiece nor tamper with the mechanism.
- Do not allow any impact on the shank. Do not drop the handpiece.
- Check the vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using immediately and contact your dealer.
- Should the handpiece function abnormally, cease operation immediately and return the handpiece to the dealer for repair.
- Supply PANA SPRAY Plus after each use and before autoclaving.
- Do not wipe with, or clean or immerse in, high acid water or sterilizing.
- Connecting /disconnecting handpiece or tip have to perform when the handpiece stopped.

1.Connecting / Disconnecting the motor and the handpiece

To connect the motor and the handpiece, align the positioning marks of the motor and the handpiece, and insert straightly. This procedure will also align the light receptacle and allow you to use illumination if your handpiece and motor are equipped for illumination. (X55L)

To disconnect, hold the front part of the motor and the back part of the handpiece, and pull out straightly.

CAUTION

- Do not connect or disconnect the handpiece until the motor has completely stopped.

2. Insertion and Removal of the Eva Tip or Profin Tip

(1) Tip Insertion

A) Eva Tip

Insert an Eva Tip into the head.

B) Profin Tip

Insert a Profin Tip into the head. After adjusting the direction of the tip to the desired angle, push it firmly into the slit at the plunger of the head. (Fig.1)

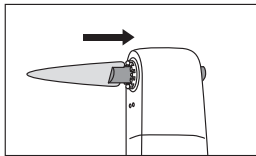
Fig. 1

(2) Removal of Tip

Insert a TIP REMOVER in the hole of the handpiece back head and begin to thrust a tip. (Fig.2).

CAUTION

Tip possibly protrudes. Remove it on the tray.

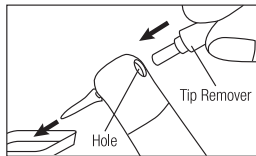
Fig. 2

3. Check before treatment

Check for handpiece vibration, noise and overheating. If any abnormalities are found do not use the handpiece and contact your Authorised NSK Dealer.

4. Lubrication

CAUTION

Use PANA SPRAY Plus and/or the NSK automatic lubrication system after each use and before each autoclaving.

■ PANA SPRAY Plus

1. Push E-type spray nozzle attachment over the PANA SPRAY Plus nozzle until it firmly seats.
2. Remove the tip from handpiece.
3. Shake the can 3-4 times to well mix lubricant and propellant.
4. Insert the E-type spray nozzle in the rear of the handpiece and spray for approximately 2-3 seconds until the oil comes out of the handpiece head.

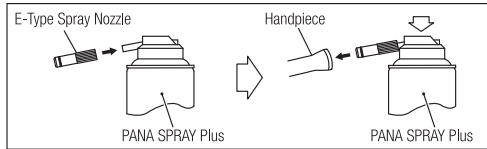
Fig. 3

CAUTION

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece head (for approx. 2 seconds).
- Keep the PANA SPRAY Plus can upright.

■ NSK automatic lubrication system

When using NSK automatic lubrication system, refer to the system instructions.

5. Sterilization

This product can be washed via Thermo-Disinfector.

NSK recommends autoclaving.

Sterilization is required first time you use and after each patient as noted below.

■ Autoclave Procedure:

1. Scrub dirt and debris from the handpiece, and wipe clean with alcohol-immersed cotton swab or cloth. Do not use a wire brush.
2. Lubricate with PANA SPRAY Plus. Refer to 4. Lubrication.
3. Insert into an autoclave pouch. Seal the pouch.
4. Autoclavable under the conditions below.

Type	Class N	Class B
Temperature	132°C	132°C
Full Cycle Time	15 min	4 min
Drying Time	30 min	30 min

5. Keep the handpiece in the autoclave pouch to keep it clean until you use it.

CAUTION

- Do not heat or cool the handpiece quickly. Rapid change in temperature could break the glass rod give abnormal strain to other metals.
- Do not wipe with, or clean or immerse in, high acid water or sterilizing solutions.

6. Cleaning Glass Rod Optic End (X55L)

In case the light had become dim, clean the glass rod optic end with an alcohol-soaked cotton swab. (Fig.4)

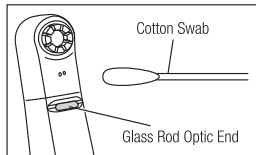
Fig. 4

CAUTION

Do not use a pointed tool or sharp edges to clean the glass rod optic end. They may damage the mirror finish and reduce the light transmission efficiency. If the glass rod optic end has become damaged, send the handpiece to the NSK dealer for repair.

7. SPECIFICATION

Model	X55L	X55
Intended to use	Interdential polishing	
Tip	EVA/Profin Tip	EVA/Profin Tip
Motion	Vertical Reciprocating (Stroke Length 1.4 mm)	Vertical Reciprocating (Stroke Length 1.4 mm)
Allowable Max. Speed	40,000min ⁻¹	40,000min ⁻¹
Gear Ratio	4 : 1 Reduction	4 : 1 Reduction
Spray Type	Single Spray	Single Spray
Optics	Glass Rod Optic	-

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NSK America Corp. www.nskdental.com