OPERATION MANUAL

Rx Only

MADE IN JAPAN

OM-DC0926EN 001

FX Contra

FX15

FX25/FX25K

FX57

1 User and Indications for Use

FX15 / FX25 / FX25K

User: Qualified Professionals (Dentist)

Indications for Use:

FX Contra is intended for the following application(s):
Caries removal, cavity and crown preparation, removal of dental restorations (fillings and prostheses), finishing and polishing of teeth and dental restorations.

FX57

User: Qualified Professionals (Dentist/Dental hygienist)

Indications for Use:

FX Contra is intended for the following application(s):
Prophylaxis treatment of the surface of teeth and dental restorations.

CAUTION

- Connect ONLY to E type motors (ISO 3964).

See the “11 Symbol” section on page 27 for reference.

2 Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
⚠ WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.

⚠ CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

⚠ WARNING

- The handpiece is delivered in a non-sterile condition and must be cleaned and sterilized prior to its first use and after each patient use.
- Clean and lubricate the handpiece immediately (within 1 hour) after each treatment to remove residue. Failure to properly maintain the handpiece may cause infection, product failure or overheating, leading to burn injuries. (Refer to “7 Post-use Maintenance”)
- If blood infiltrates inside a handpiece, clean and lubricate the handpiece immediately using the PANA SPRAY Plus.

⚠ WARNING

- Depressing the push button while the handpiece is in rotation may lead to overheating, causing burn injuries or product failure. Avoid the push button to contact with any oral tissue. (Except for FX57)
- Cool down the cutting site with a dental air/water syringe while using the handpiece. Insufficient cooling may lead to overheating of the cutting site, causing burn injuries. (Except for FX57)
- Use polishing paste only FX57. When polishing paste is used other handpieces, the polishing paste could enter the handpiece, causing failures of the ball bearing or burn injuries due to the overheating of the handpiece head.

WARNING

- Keep any debris or other foreign materials away from inside the gear or the handpiece. Foreign materials remaining inside may lead to overheating, causing burn injuries or other accidents.
- Ball bearings are wear-and-tear items and may require replacement. Be sure to inspect the handpiece before use (Refer to “6 Checking the Handpiece Before Each Use”). If abnormal vibration or noise is found, the bearings or other internal parts may be worn out or damaged. Damaged parts may lead to handpiece overheating causing burn injuries. If overheating is suspected, stop using the handpiece immediately and contact your Authorized NSK Dealer.

CAUTION

- Read this operation manual before use to fully understand the product functions and file for future reference.
- When operating the product, always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Prior to clinical use, inspect the handpiece. Check for vibration, noise and overheating. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer. (Refer to “6 Checking the Handpiece Before Each Use”)
- Do not disassemble or alter the handpiece except as recommended by NSK in this operation manual.
- Do not allow any impact to the product. Do not drop the product. Deformation may cause the handpiece to fail during use.

⚠ CAUTION

- Never use counter clockwise. It may run out the rubber cup/brush and cause an injury. (FX57)
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.
- Do not use burs/cups/brushes with problems listed below as the bur/cup/brush may break, seize up or disengage from the chuck/screw hole.
 - Bent, deformed, worn, rusted, broken, deficient bur/cup/brush.
 - Bur which is cracked on the edge or axis.
 - Non-ISO standard, or tampered bur.
- Do not use burs/cups/brushes other than specified in "10 Specifications" as such burs/cups/brushes may be accidentally released or break during rotation.

⚠ CAUTION

- Always keep the bur shank/cup and brush screw thread clean. Dirt or debris in the chuck/screw hole could cause poor bur/cup/brush concentricity or low chuck/screw hole retention force.
- Always insert the bur/cup/brush all the way into the chuck/screw hole. If insertion is insufficient, premature failure of the bearings or accidental release of the bur/cup/brush may occur.
- Do not exceed Max. Bur Length shown in "10 Specifications".
- Always follow the instructions provided by the bur/cup/brush manufacturer.
- Do not exceed the bur/cup/brush speed recommended by the bur/cup/brush manufacturer.
- The maximum surface temperature of the handpiece might reach 48°C (118 °F) depending on use.

CAUTION

- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized NSK Dealer.
- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.
- Perform regular function and maintenance checks. (Refer to “9 Periodical Maintenance Checks”)
- If the handpiece has not been used for a long period, rotate the handpiece and check for noise, vibration and overheating before use.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.

CAUTION

- The operation of the handpiece is permitted only on dental units, which correspond to the standards IEC 60601-1 and IEC 60601-1-2.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

NOTICE

- Repairs of this product are only to be performed by authorized service technicians according to NSK requirements. Contact your Authorized NSK Dealer if repairs are necessary.

3 Package Contents

No.	Part Name	Quantity
1	E-Type Spray Nozzle	1
2	FX Wrench*	1

*Included in the package for FX25 / FX25K

⚠ CAUTION

- E-Type Spray Nozzle and FX Wrench cannot be sterilized.

4 Connecting and Disconnecting the Handpiece from the Motor

4-1 Connecting

- 1 Insert the handpiece directly into the motor (Fig. 1).
- 2 Confirm that the handpiece is firmly connected to the motor.

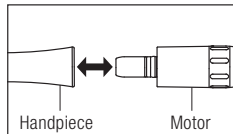
Fig. 1

4-2 Disconnecting

Hold the motor and the handpiece, then pull apart (Fig.1).

⚠ CAUTION

- Do not connect or disconnect the handpiece until the motor has completely stopped.
- Do not exceed the Max. Rotation Speed (Motor) shown in “10 Specifications”.
- Connect ONLY to E type motors (ISO 3964).
- Make sure that the motor and handpiece are securely connected. The pressure of handpiece cooling air from the motor may cause the handpiece to jump out of the motor, which could result in injury.

5 Mounting and Removing the Bur/Cup/Brush

5-1 Mounting the Bur (FX15 / FX25 / FX25K)

- ① Insert the bur until it is correctly seated in place. (Fig. 2)
- ② Depress the push button and insert the bur into the chuck until the bur “notch” mechanism engages. Release the button.
- ③ Confirm that the bur is not removed by gently pulling and pushing the bur without depressing the push button.

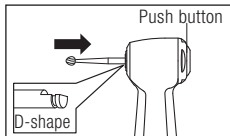
Fig. 2

5-2 Removing the Bur (FX15 / FX25 / FX25K)

Depress the push button to open the chuck (1) and remove the bur (2). (Fig. 3)

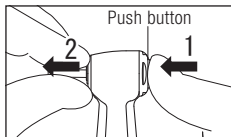
Fig. 3

5-3 Mounting the Cup/Brush (FX57)

- 1 Insert the screw shank of the cup or brush into the screw hole of the handpiece head.
- 2 Turn cup/brush in a clockwise direction to fix the screw shank.
- 3 Check that the screw part of the cup or brush is inserted to the end. (Fig. 4)

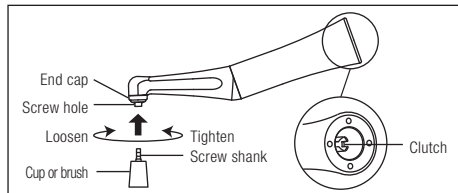
Fig. 4

5-4 Removing the Cup/Brush (FX57)

Remove the handpiece from the motor, hold the cup or brush firmly while the handpiece clutch is fixed, and turn it in the loosening direction as shown in Fig. 4.

WARNING

- Always use a clean, sterile bur/cup/brush. Be sure to wear sterile gloves before mounting the bur/cup/brush.
- When changing the bur/cup/brush during treatment, remove the bur/cup/brush and wipe around the bur insertion hole or screw hole with a dry lint free cloth until it is visually clean, then mount a clean and sterile bur/cup/brush. Residual blood or debris may enter the handpiece while changing bur/cup/brush, which could lead to higher risk of cross-contamination.

CAUTION

- Do not use burs/cups/brushes with problems listed below as the bur/cup/brush may break, seize up or disengage from the chuck/screw hole.
 - Bent, deformed, worn, rusted, broken, deficient bur/cup/brush.
 - Bur which is cracked on the edge or axis.
 - Non-ISO standard, or tampered bur.
- Do not use burs/cups/brushes other than specified in “10 Specifications” as such burs/cups/brushes may be accidentally released or break during rotation.

⚠ CAUTION

- Insert a sterilized bur/cup/brush and always keep the bur/cup/brush shank clean. When changing bur/cup/brush, make sure that there is no dirt or debris in and around the bur/cup/brush insertion hole of the handpiece. Debris in the chuck/screw hole could cause infection, heating, poor bur/cup/brush concentricity or low chuck/screw hole retention force.
- Always insert the bur/cup/brush all the way into the chuck/screw hole. If insertion is insufficient, premature failure of the bearings or accidental release of the bur/cup/brush may occur.
- Do not exceed Max. Bur Length shown in “10 Specifications”.
- Always follow the instructions provided by the bur/cup/brush manufacturer.

⚠ CAUTION

- Do not exceed the bur/cup/brush speed recommended by the bur/cup/brush manufacturer.
- Do not mount or remove the bur/cup/brush until the motor has completely stopped.
- Do not apply excess pressure to the bur/cup/brush as it may break or bend or become difficult to remove.

6 Checking the Handpiece Before Each Use

Follow the check below before use. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer.

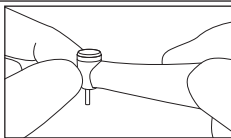
Fig. 5

- 1 Check the head cap is firmly tightened.
- 2 Mount the bur/cup/brush and make sure that it is securely connected. (Refer to “5 Mounting and Removing the Bur/Cup/Brush”)

- 3 Rotate the handpiece for about one minute at the Max. rotation speed of the attached motor.
During rotation, check for abnormalities such as abnormal rotation, vibration, noise.
- 4 After the handpiece rotation has completely stopped, touch the handpiece head to confirm it is not heating abnormally (Fig. 5).

⚠ CAUTION

- To avoid injury, keep your hands away from the bur during rotation.
- If the bur/cup/brush connection is loose, the part may fall out, which could result in accidental ingestion.
- Heat generation of the handpiece may cause burn injuries.
- If the head cap or end cap is loose, the part may fall out, which could result in accidental ingestion.

7 Post-use Maintenance

After each patient, maintain the product as follows.

⚠ WARNING

- To ensure cleaning and sterilization efficacy, use only the following procedures for reprocessing.
- Follow any additional local directives, standards, and guidelines for cleaning and sterilization.
- The handpiece is delivered in a non-sterile condition and must be cleaned and sterilized prior to its first use and after each patient use.
- Clean and lubricate the handpiece immediately (within 1 hour) after each treatment to remove residue. Failure to properly maintain the handpiece may cause infection, product failure, overheating leading to burn injuries.

7-1 Cleaning at point-of use

⚠ CAUTION

- Do not use the following liquids to wipe, immerse, or clean the product: strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzene or thinner.
- Do not immerse NSK instruments in disinfectant solution or clean in ultrasonic devices.

- 1 Always wear protective gloves, a mask, and protective goggles for safety purposes and to minimize the risk of infection. (Fig. 6)

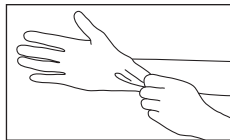
Fig. 6

- ② Remove the bur/cup/brush. (Fig. 7)
- ③ Wipe the exterior of the handpiece clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90%. (Fig. 8)

When using disinfectant, follow the instructions given by the manufacturer of the disinfectant.

Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties. The following disinfectants can be used in the United

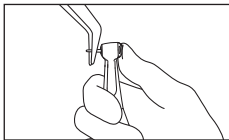
Fig. 7

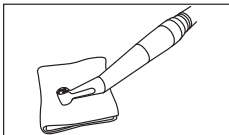
Fig. 8

States and Canada.

CaviCide®, CaviWipes® (manufactured by Metrex)

- ④ Operate the handpiece at the chair-side for at least 20 seconds. After the handpiece rotation has completely stopped, touch the handpiece head to confirm it is not heating abnormally.

⚠ CAUTION

- If the handpiece head is heating abnormally, stop using the handpiece immediately and contact your Authorized NSK Dealer.

- ⑤ Remove the handpiece from the motor. (Fig. 9)
- ⑥ Carry the handpiece to the decontamination area.

Fig. 9

7-2 Cleaning and Drying

Wash the handpiece under running water following the procedures below.

(Water condition: $\leq 38^{\circ}\text{C}$ (100 °F), $\geq 3.5\text{L/min}$; water should be of the same quality as drinking water)

This product can not be cleaned and disinfected with a washer-disinfector.

- 1 Clean the external surfaces of the handpiece with a soft bristled toothbrush for 15 seconds or more. (Fig. 10)

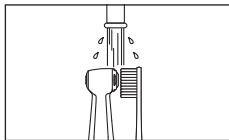
Fig. 10

- 2 Clean around the bur insertion hole for 15 seconds or more, using an interdental brush (wire thickness 0.7mm) that conforms to size 4 in ISO 16409. (Fig. 11)

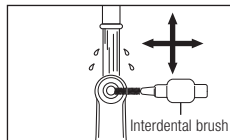
Fig. 11

Under appropriate lighting (natural office lighting, 500 lx or higher), inspect the handpiece for blood or other debris. If any visible debris remains, repeat the process until the handpiece is visually clean.

Optional

Wipe the exterior of the handpiece, clean using a cloth moistened with disinfectant or ethanol with a concentration of 60-90%. (Fig. 12)

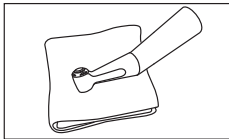
Fig. 12

NOTICE

- Wiping is not an alternative method of cleaning. Please do not skip the required cleaning steps which are previously displayed.

When using disinfectant, follow the instructions given by the manufacturer of the disinfectant.

Use a state-sanctioned disinfectant with proven bactericidal, fungicidal, and virucidal properties.

The following disinfectants can be used in the United States and Canada.

CaviCide®, CaviWipes® (manufactured by Metrex)

After cleaning, if residual moisture is present, wipe off with dry cloth or blow it off with compressed air ($\leq 0.35\text{MPa}$) until there is no moisture in the interior and exterior.

When blowing off with compressed air, cover the handpiece with cloth to prevent scattering of water. Proceed to “7-3 Cleaning and Lubrication”.

7-3 Cleaning and Lubrication

- 1 Attach an E-Type Spray Nozzle to the nozzle of the PANA SPRAY Plus.
- 2 Shake the can 3 or 4 times, and insert the E-Type Spray Nozzle into the rear of the handpiece while holding the handpiece head with a cloth etc.
- 3 Hold the handpiece, and spray for 2 or 3 seconds until oil comes out of the handpiece head (Fig. 13). Repeat lubrication until foreign material stops coming out of the tip.

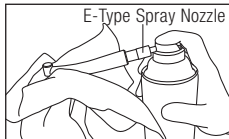
Fig. 13

⚠ CAUTION

- Do not use sprays other than NSK PANA SPRAY Plus. Using sprays other than PANA SPRAY Plus may cause the handpiece to overheat.
- Hold the spray can upright.
- Firmly hold the handpiece to prevent it slipping when spray pressure is applied.
- Spray lubricant until it expels from the handpiece head.
- If you wish to purge excessive oil from inside the handpiece, rotate the handpiece for approx 15 seconds without a bur. During the rotation, do not depress the push button, especially when using absorbent cloth to prevent oil from scattering. Depressing the push button during rotation may damage the chuck mechanism (Fig. 14, Fig. 15).

⚠ CAUTION

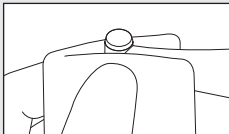
Fig. 14

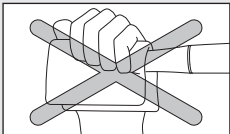
Fig. 15

⚠ WARNING

- If blood infiltrates inside a handpiece, clean and lubricate the handpiece immediately using the PANA SPRAY Plus.

NOTICE

- NSK recommends the use of “Spray Mist Absorber” (**REF** Y900084) to prevent oil mist expelling out of the handpiece head.

7-4 Packaging, Sterilizing, and Drying

- 1 Insert the handpiece, into an FDA-approved sterilization pouch (es) that conforms to ISO 11607-1, and seal the pouch (es).
- 2 Perform steam sterilization with the following conditions.

Type	Gravity Displacement	Pre-Vacuum (Dynamic Air Removal)
Temperature	132°C (270 °F)	132°C (270 °F)
Full Cycle Time	15 min	4 min
Drying Time	30 min	30 min

CAUTION

- Use an FDA-approved steam sterilizer to perform sterilization.
- Follow local rules, regulations, and guidelines regarding the reprocessing of devices.
- Immediately after sterilization, the product is hot and should be handled with caution.
- Place each product in an individual sterilization pouch for steam sterilization. This is to prevent possible discoloration and damage to the product due to chemical residue from other instruments.
- Clean and lubricate the handpiece prior to sterilization. If blood remains on the internal surface it can become clotted and cause product failure.

CAUTION

- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- Be sure to use sterilizers that can perform sterilization up to 135°C (275 °F). In some sterilizers, the chamber temperature may exceed 135°C (275 °F). Do not use these sterilizers as failure of the handpiece could occur. Contact the sterilizer manufacturer for detailed information about cycle temperatures.
- Steam sterilization is recommended for the product. The validity of other sterilization methods (such as plasma sterilization or EOG sterilization) is not confirmed.

7-5 Storage

Store the product in a dry, clean location.

CAUTION

- After the sterilization and drying cycles are complete, remove the handpiece immediately from the sterilizer to store it.
- Store the product in a well ventilated place out of direct sunlight and within the range of temperature, humidity and pressure specified in “10 Specifications”.
- Sterilization is not guaranteed after the sterilization retention period specified by the manufacturer and seller of the sterilization pouch has elapsed. If the sterilization retention period has elapsed, perform sterilization again with a new sterilization pouch.

8 Replacing the Cartridge (FX25 / FX25K)

Replace the cartridge if any of the following symptoms occur:

- Abnormal rotation, vibration, or noise
- Abnormal heating of the handpiece head
- Loose bur connection

❶ Perform the steps from “7–1 Cleaning at point-of use” to “7–4 Packaging, Sterilizing, and Drying”.

❷ Detach the shank by turning it counter clockwise and remove the handpiece head from the shank (Fig. 16).

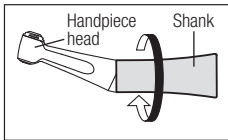
Fig. 16

⚠ CAUTION

- After removing, do not hold the shank upside down as it could cause the internal parts to drop.
- Be sure to perform the steps from “7–1 Cleaning at point-of use” to “7–4 Packaging, Sterilizing, and Drying” to prevent infection.

❸ Insert the FX Wrench firmly into the rear of the handpiece head until it stops, as shown in Fig. 17.


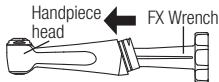
Fig. 17

Fig. 18



- ④ Tightly hold the FX Wrench as illustrated in Fig. 19 then pull the drive shaft straight out in the direction of the arrow (Fig. 19).

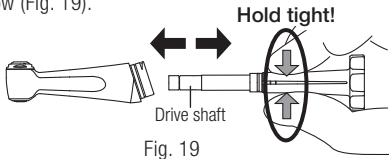⚠ CAUTION

- Be sure to insert the FX Wrench straight into the handpiece head. Do not insert the FX Wrench at an angle as shown in Fig. 18.
- When removing the drive shaft, tightly hold the FX Wrench as directed in Fig. 19.

- ⑤ Locate the correct FX Wrench for the head cap then turn the FX Wrench counter clockwise to loosen the head cap. Remove the head cap (Fig. 20).
- ⑥ Use the bur to gently push the cartridge and washer out from the handpiece head (Fig. 21).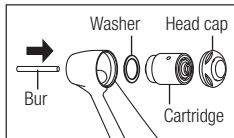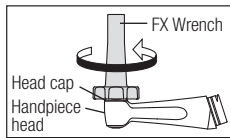

- ⑦ Clean the handpiece head interior with NSK PANA SPRAY Plus.
- ⑧ Wipe the NSK PANA SPRAY Plus oil over the handpiece head interior.

- 9 Make sure that there are no old washer or foreign objects in the handpiece head before inserting a new washer (Fig. 22).

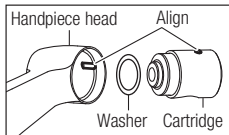
Fig. 22

- 10 Insert the new cartridge into the handpiece head by aligning the pin on the cartridge with the slot on the handpiece head (Fig. 22).
- 11 Tighten the head cap with the correct FX Wrench.
- 12 Insert the drive shaft firmly into the handpiece head until it clicks. If the click is not heard, insert it again.
- 13 Reattach the shank.

*Refer to “12–3 Spare Parts List” to identify the correct cartridge.

- 14 Clean and sterilize the handpiece after replacing the cartridge.

⚠ CAUTION

- The FX Wrench should only be used on a reprocessed handpiece and outside of the clinical operatory space.
- Before turning the head cap, make sure the FX Wrench is correctly aligned with the head cap. Do not turn the FX Wrench at an angle as abrasion of the FX Wrench may occur.
- Secure the shank firmly to the handpiece head so that there is no gap between them. A loose connection could make the handpiece head unstable, resulting in an accident during use.
- Use only a genuine NSK cartridge. If a non genuine cartridge is used NSK cannot guarantee performance and the handpiece warranty would become invalid.
- Always first finger tighten the head cap then secure firmly with the FX Wrench.

CAUTION

- NSK never recommends the disassembly and repair of any NSK cartridge. There is NO EXCEPTION. In such a case a handpiece may perform abnormally (abnormal noise or abnormal vibration). Damage, failure or accidents are outside of our guarantee.

9 Periodical Maintenance Checks

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check

Details

Head cap is loose	Check that the Head cap is firmly tightened.
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.
Connection	Check the connection between the cup or brush, motor and handpiece for any problems.

10 Specifications

EN

FR

Model	FX15	FX25	FX25K*
Handpiece Connection Type	ISO 3964 Type1		
Max. Rotation Speed (Motor)	40,000min ⁻¹		
Max. Rotation Speed (Handpiece)	10,000min ⁻¹	40,000min ⁻¹	
Gear Ratio	4:1 Reduction	1:1 Direct Drive	
Bur Type	ISO 1797 Type1 Ø2.334-2.35mm CA Bur		
Chuck Length	12.7mm		
Max. Bur Length	22.5mm		
Max. Working Part Diameter	Ø4.0mm		

*This product can be connected to a motor with Ø16mm outer diameter.

Model	FX57
Handpiece Connection Type	ISO 3964 Type1
Max. Rotation Speed (Motor)	20,000min ⁻¹
Max. Rotation Speed (Handpiece)	5,000min ⁻¹
Gear Ratio	4:1 Reduction
Rotary Instrument Type	Screw-in cup/brush (Screw thread in accordance with ISO 13295 Type 5 M1.8×0.35 (No.1-64UNC / No.1-72UNF))
Max. Rotary Instrument Length	16.0mm
Max. Working Part Diameter	Ø7.0mm

Use Environment	Temperature: 10 - 35°C (50 - 95 °F), Humidity: 30 - 75% (No Condensation)
Transportation and Store Environment	Temperature: -10 - 50°C (14 - 122 °F), Humidity: 10 - 85%, Pressure: 500 - 1,060hPa

11 Symbol

This product can be sterilized in a steam sterilizer at 135°C (275 °F).

Conforms to CE European Directive of "Medical device directive 93/42/EEC."

Manufacturer

Date of manufacture

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

GS1 DataMatrix for Unique Device Identifier.

See operation manual.

Caution, Consult operation instructions.

Temperature limitation

Humidity limitation

Atmospheric pressure limitation

Unique device identification

REF

Catalog number (Order code)

SN

Serial No.

CAN

Device Name for Health Canada.

12 After-sales Service

12-1 Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

Contact your Authorized NSK Dealer if repairs are necessary.

12-2 Option List

Model	REF	Handpiece Model	Remarks
PANA SPRAY Plus	Z182600	All models	-

12-3 Spare Parts List

Model	REF	Handpiece Model	Remarks
FX25-03	C1052700	FX25 / FX25K	Cartridge with washer
FX25-04	C1052500	FX25 / FX25K	Head cap
FX25-02	10000946	FX25 / FX25K	Drive shaft

12-4 Accesories

Model	REF	Handpiece Model	Remarks
FX Wrench	C1052701	FX25 / FX25K	-
E-Type Spray Nozzle	Z019090	All models	-

13 Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled medical wastes, to dispose the product for you.

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NSK America Corp. www.nskdental.com